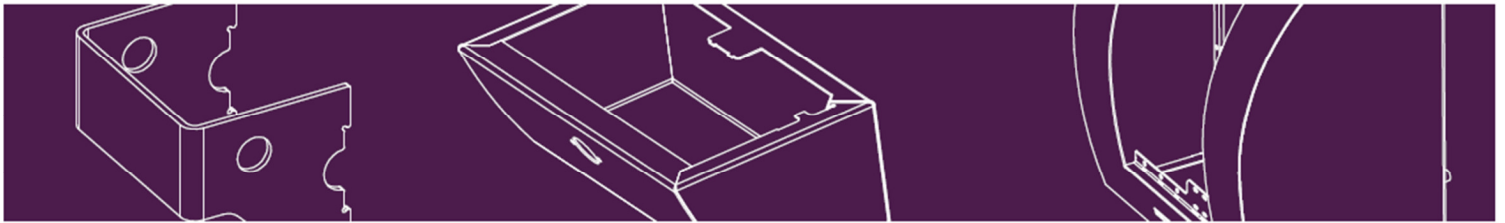
## SmartCase SC1230 / SC1240

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
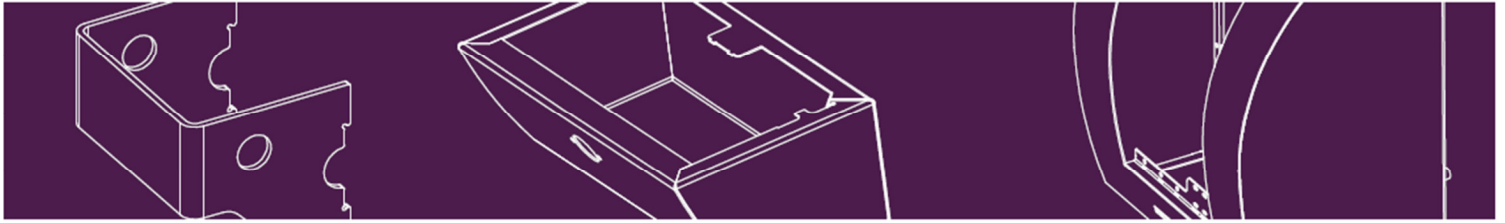
## SmartCase SC1230 / SC1240

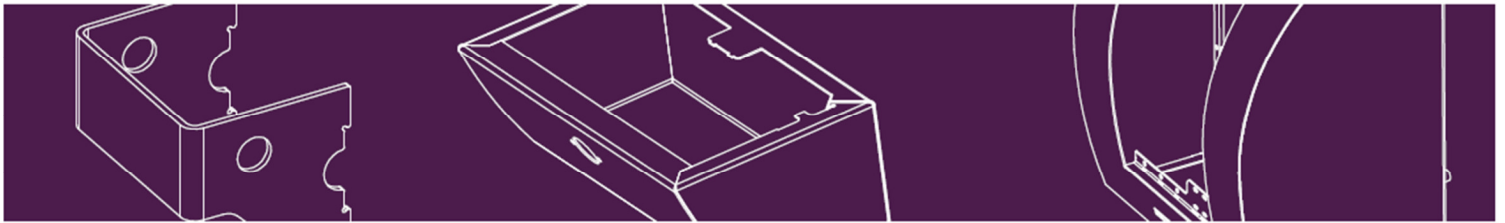
## Table of Contents
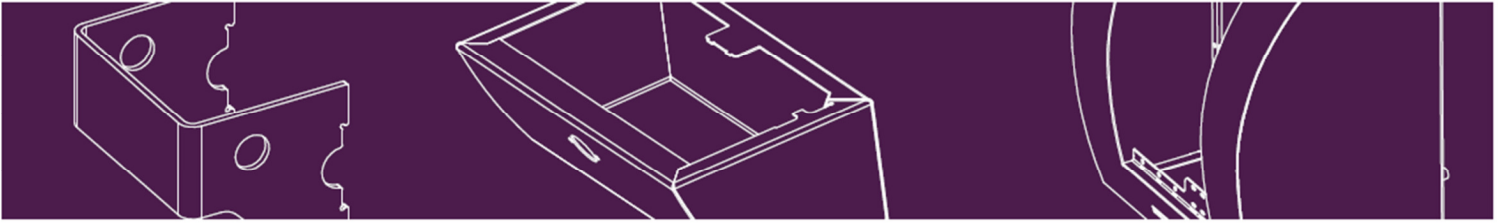
<b>WHAT IS A SMARTCASE? .....</b>	<b>3</b>
<b>FOR YOUR SAFETY .....</b>	<b>3</b>
CONVENTIONS: .....	3
IMPORTANT: .....	3
<b>WARRANTY .....</b>	<b>4</b>
<b>PRODUCT OVERVIEW .....</b>	<b>5</b>
WHAT DOES THE SMARTCASE CONTAIN? .....	5
.....	5
<b>INSTALLATION .....</b>	<b>6</b>
IMPORTANT: .....	6
MOUNTING ON BRICK OR CONCRETE: .....	6
FEEDER CABLE: .....	6
BE CAREFUL: .....	7
MOUNTING ON WOOD: .....	7
<b>OPERATION .....</b>	<b>7</b>
IMPORTANT: .....	7
<b>MAINTENANCE AND TROUBLESHOOTING .....</b>	<b>8</b>
MAINTENANCE: .....	8
TEMPERATURE: .....	8
CHANGING CODE (SC1240) .....	9
TROUBLESHOOTING: .....	10
<b>GLOSSARY .....</b>	<b>11</b>
<b>APPENDIX A: TECHNICAL SPECIFICATION .....</b>	<b>12</b>

## What is a SmartCase?

The AED (Automatic External Defibrillator) is an important device for primary help in case of a cardiac arrest. It is of a vital importance that the AED is protected, secured and available 24/7 for Immediate assistance in case of an emergency. Your SmartCase cabinet provides the ultimate protection and assures day and night availability of the AED.

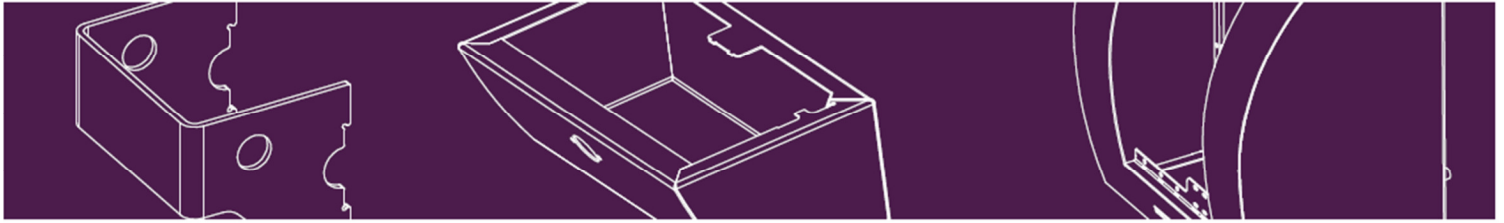
The SmartCase outdoor cabinet is designed to protect the AED against extreme weather conditions. The cabinet resists water and dust, violence and combines a proven concept with maximum performance. The stainless steel box combined with a polycarbonate window provides the ultimate protection against corrosion and impacts and offers a clear sight of the AED. To maintain a temperature above 0°C, the outdoor SmartCase is equipped with a temperature sensor and electrical heater. The cabinet is fully insulated to save energy during heating. Additional protection against overheating is achieved by active ventilation.

## For your safety

**Conventions:** Read the user manual carefully. All operators should review these safety considerations before placing the SmartCase AED Cabinet into service. Not following the instructions may be dangerous or illegal. Follow all recommended maintenance instructions. If a problem occurs, obtain service immediately. Do not use the device until it has been inspected by appropriate personnel.


		Symbol
<b>Warning:</b>	Warning statements alert you to conditions or actions that can result in personal injury or death.	 WARNING

**Important:** GT MEDICARE is neither responsible for the contents of this document nor for the loss of data, income, direct or indirect caused damage. GT MEDICARE preserves the right to adapt and to improve this document and all related products in this user manual, without preceding notification. The contents of this document are provided without any form of warranty.

## Warranty

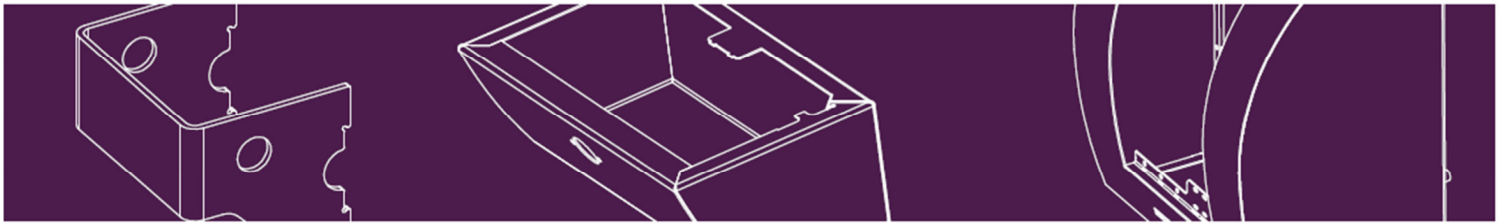1. GT MEDICARE is responsible for the quality of services and for the soundness and good quality of the products supplied to the best of its knowledge, with the provision that the guarantee does not exceed the following provisions mentioned.
2. If the defects are caused by purchaser the repair and shipping costs will be charged to purchaser.
3. The purchaser is not entitled to warranty:
  - A: If the buyer has neglected the products.
  - B: If the buyer has made changes to the products or repairs which were not performed by or on behalf of GT MEDICARE.
  - C: In case of improper or careless use, incorrect connection, wrong voltage, lightning, damage due to moisture or other external products or misfortunes.
  - D: If the device is not appropriately maintained according to the manual.
  - E: If the device is used or installed with non original or inferior accessories.
  - F: If the buyer dealt with the goods with negligence.
4. GT MEDICARE warrants that the Products do not contain serious defects at the time of delivery.
5. Reseller will report defects, damages or non-conformity of the Products, which upon inspection can reasonably be determined, in writing to GT MEDICARE within 10 days after reception. Reseller will report all other defects in writing to GT MEDICARE within one year from the date of delivery. Products, which suggest that they are non-conforming or damaged, should be returned to GT MEDICARE immediately for inspection by GT MEDICARE. Claims, which were not reported according to the terms in this paragraph, will not be accepted. GT MEDICARE will, in case of responsibility for the non-conformity or damage to the product, at its option, repair the Products, replace them with new ones or refund the purchase price of the Products. GT MEDICARE will have no other obligation or liability towards Reseller with respect to non-conforming or damaged products other than in the previous sentence displayed.
6. The duration of the warranty on all mechanical parts is 60 months, with scratches and damage to the cabinet and window outside this provision, and 24 months for the electronics from the time of delivery. The producer is only obliged to fulfil the obligations arising from this warranty. The producer is not liable for indirect or consequential damages. In all cases, the manufacturer's liability is limited to the selling price of the products sold.
7. Where a claim can be attributed to a modification or improper use of the product, the guarantee is under all circumstances void.
8. The recipient will investigate all received equipment immediately. The customer must inform GT MEDICARE in writing within ten (10) days after receipt of any defects in the consignment. If the deficiencies are not reported within a period of ten (10) days, the shipment was considered to be complete. For hidden, to prove by customer, defects, claims are only possible within the warranty period. If a complaint is justified by GT MEDICARE, then GT MEDICARE is only obliged to replace the defective goods or repair, without the customer being entitled to claim any compensation whatsoever. Submitting a claim does not relieve the customer from payment.

## Product Overview

Thank you for purchasing a SmartCase.

**What does the SmartCase contain?** The SmartCase contains the following parts:

### SmartCase SC1230 and SC 1240

- SmartCase AED Wall Cabinet (insulated)
- 24V Adapter
- Feeder cable 3m (pre-installed)
- Quick Installation guide
- Electrical heater & heater bracket
- Fan for active ventilation
- LED lights (red and blue)
- Alarm, >95dB
- Installation materials

### Installation materials:

Washers (4x)

M8 Nuts (4x)

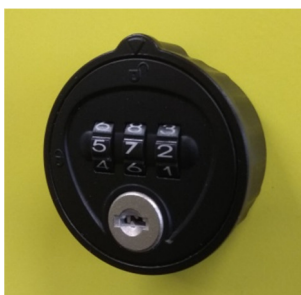
Nylon spacers (4x)

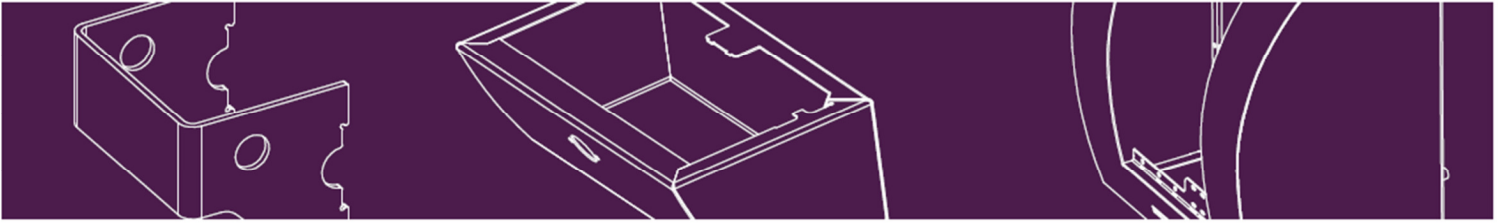
Wall plug (4x)

Dowel screw (4x)

### SmartCase SC1240

- Key
- The SC 1240 is equipped with a mechanical lock. For this version an additional back-up key is included

## Installation

**Important:** The installation tools supplied are intended for mounting exclusively on flat surfaces of wood, brick or concrete. Panels must have a minimum thickness of 3mm. Always use the supplied spacers. We recommend installing the cabinet in the shadow to prevent high temperatures inside the cabinet.

**Mounting on brick or concrete:** Drill 4 holes for the attachment bolts (dowel screws) with a 240 mm spacing. Drill the round 10 mm holes to a minimum depth of 60 mm, to ensure there is sufficient depth for the plug. We recommend drilling the upper holes at between 1.5 and 1.6 metres from the ground. The holes should always be drilled in the middle of the brick. Insert the M8 dowel screws in the plugs until they protrude 20mm from the wall and slide the nylon spacers over the dowel screws. Before mounting the cabinet on the wall, first install the feeder cable.

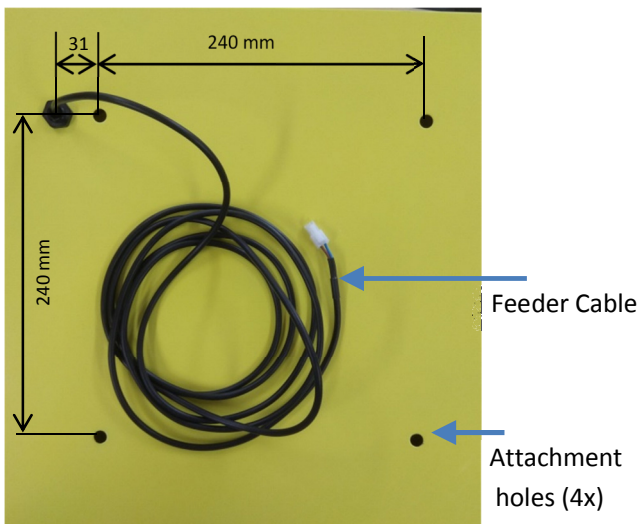
1: Insert Plug and dowel screws

2: Place spacer


**Feeder cable:** GT MEDICARE recommends drilling the 12 mm hole in such a way that the feeder cable can be put led through the wall. It can then be connected to the 24V adapter, which should be placed inside the house or building. It is important to lead the feeder cable through the wall before mounting the case on the wall. The cable has been connected to the SmartCase inside the cabinet (factory pre-installed). Guide the cable from the outside through the wall to the inside and then, from the inside, pull the cable. The SmartCase is now ready to be mounted on the wall. After mounting it, use the four washers and nuts to secure the cabinet (on the inside of the cabinet).

Last step: placement of washer and M8 Nut on the inside

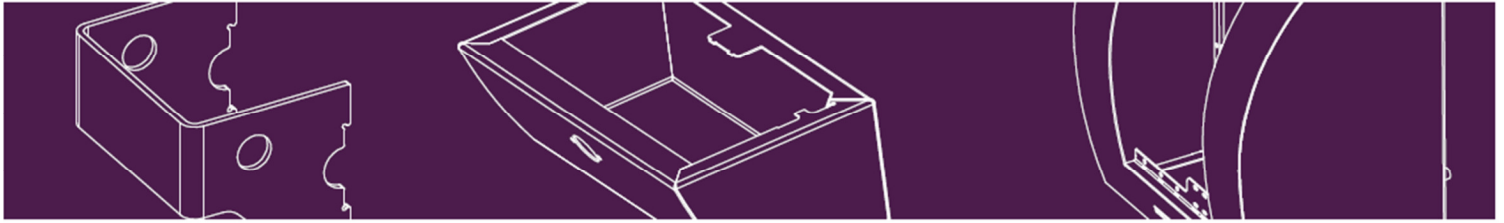Signature operator \_\_\_\_\_

**Be careful:** If the wall is not very strong, we recommend to use Fischer VS 100P. After spraying the fluid into the holes, insert only the screws – do not use the plugs again. Read the instructions on the Fischer label carefully and make sure that you clean the holes thoroughly after drilling.

**Mounting on wood:** Drill small holes of approximately 4 mm diameter in the wood and screw the wiretap firmly into the wood and then mount the SmartCase as instructed above. We recommend drilling the upper holes at a height of between 1.5 and 1.6 m from the ground.

## Operation

SC1230 (unlocked): To close the cabinet, close the door and turn the knob left (90 degrees). To open the door turn the knob in the opposite direction (90 degrees right turn) and gently pull.

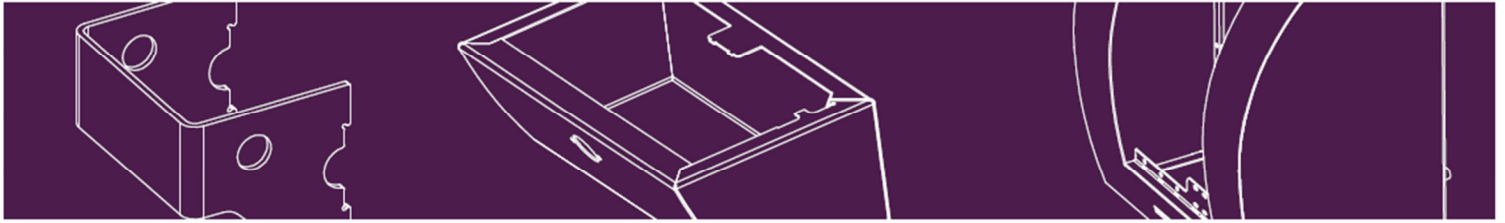
SC1240 (code lock): Enter the programmed code (default code = 333) by turning the number wheels and turn the lock to the right. In case the code is lost, you can use the key to find the code: see section **changing code**. After you have retrieved the code, you can open the lock.

Code change: see maintenance and troubleshooting

WARNING

**Important:** It is mandatory to always have the code and key available in case of an emergency (applies to SC1240). Due to an unexpected defect or wrongly programmed code, it may happen that the lock doesn't open when using the code. In this case it is necessary to have an alternative available (key). Furthermore, it is important to rehearse the above mentioned procedures before storing the defibrillator, and frequently after having installed the AED.

## Maintenance and Troubleshooting

**Maintenance:** It is vital to open and close the SmartCase (SC1240 only) wall cabinet **monthly** to check that the lock system is working correctly. While doing so, you should take careful note of the status of the defibrillator (please check manual of AED).

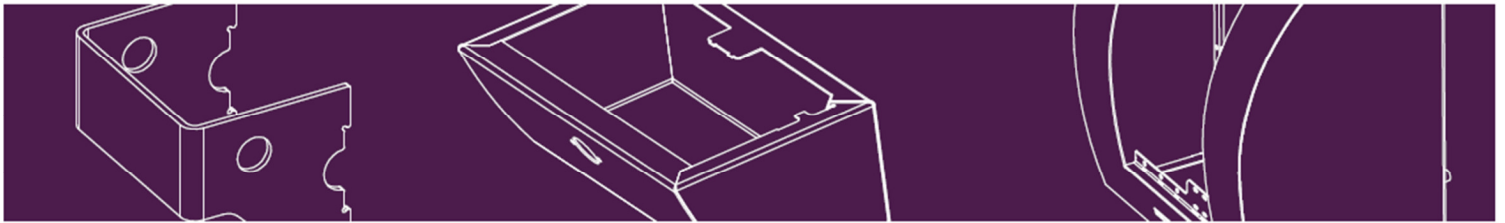
WARNING

It is also important to check the external condition of the cabinet for any damage, which may possibly affect its operation.

Upon discovering a defect, please contact your dealer of SmartCase. Depending on the type of defect it may be advisable to remove the AED and inform the users about the status (unserviceable).

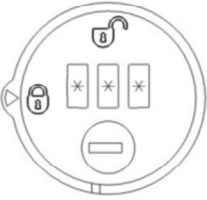
**Temperature:** If you have any doubt about the functioning of the electrical heater (outside temperature lower than 0°C / 32°F), please remove the defibrillator and contact your dealer of SmartCase. The SC1230/ 1240 are equipped with an active ventilation system. Around 30°C the fan will be switched on to release the heat from the upper part and taking in cold air from the bottom. Nevertheless, we recommend installing the cabinet in the shadow to prevent high temperatures (above 50°C).



Changing code (SC1240)

Security advise	<ol style="list-style-type: none"> <li>1. On delivery the combination lock is factory set at 333.</li> <li>2. For safety reasons set the lock immediately to your own personal code.</li> <li>3. <b>Never leave the code wheels set on your own personal code.</b></li> <li>4. Personal code must be set to open the lock and before closing it.</li> <li>5. In case of lost code the lock can be opened via service key. Please contact the supervisor to open and reset the lock..</li> </ol>
-----------------	---

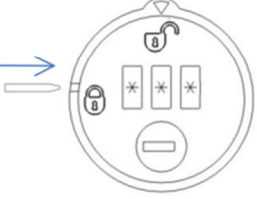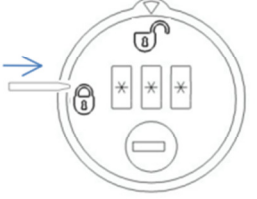Lock opening	  <ol style="list-style-type: none"> <li>1. On first use enter the factory set code 333. Otherwise use your own personal code (see point „Code setting“)</li> <li>2. Turn lock by 90° to  (Lock is then opened)</li> </ol>
--------------	--


Code setting	  <ol style="list-style-type: none"> <li>1. In <u>open-position</u> and with set code, enter a small solid pin (e.g. paper clip) into the code setting aperture on the side of the lock. Push the pin in fully and hold <u>the pressure</u>.</li> <li>2. Enter new personal code</li> <li>3. Remove code setting tool. (personal code is set!)</li> </ol>
--------------	--

Emergency opening / Code Finding	  <ol style="list-style-type: none"> <li>1. Enter service key into the barrel. Turn key anticlockwise</li> <li>2. Turn code wheels until they block (Code has been found)</li> <li>3. Turn service key back and remove it.</li> </ol>
----------------------------------	--

General information	<ol style="list-style-type: none"> <li>1. Do not expose lock to environmental influences such as water, moisture, dust, fire, etc.</li> <li>2. Lock operable within a temperature range from 0-50°.</li> <li>3. Use clean damp cloth. Do not use any chemical products.</li> <li>4. Take care that cam moves freely and is not obstructed by any item while operating the lock.</li> </ol>
---------------------	--

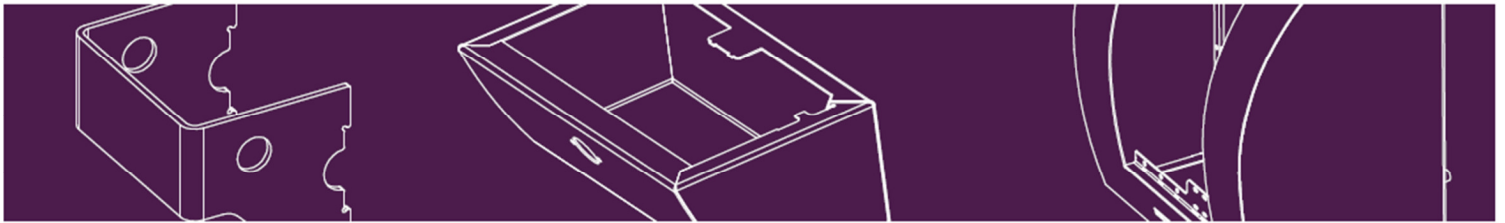
Always verify the code change by opening the SmartCase AED cabinet using the new code.

### *Troubleshooting:*

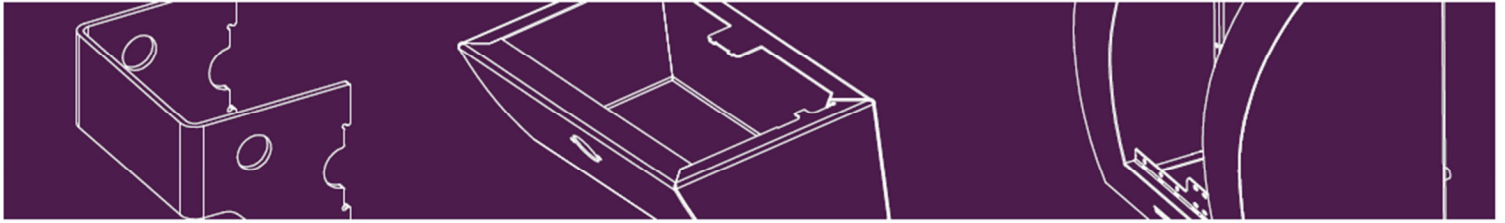Symptom	Action
<b>LED Lights are out</b>	<ul style="list-style-type: none"> <li>- Check the power supply and adapter: the LED on the adapter should light up continuously (a flashing LED or no light indicates a malfunction)</li> <li>- Make sure that all connectors are properly connected (including the connection inside the cabinet)</li> </ul> <p>If the unit continues to fail, take it out of service and contact the dealer of SmartCase.</p>
<b>Lock doesn't open when using the programmed code</b>	<ul style="list-style-type: none"> <li>- Re-program the code in accordance with this manual</li> <li>- Try to close and open the lock with the new code</li> </ul> <p>If the lock continues to fail contact the dealer of SmartCase.</p>
<b>Cabinet is not closing correctly</b>	<ul style="list-style-type: none"> <li>- Check if anything is blocking the door.</li> <li>- Check if the lock or door is deformed in the area of the locking mechanism</li> </ul> <p>If the unit can't be repaired, take it out of service and contact the dealer of SmartCase.</p>

## Glossary

### AED

The AED (Automatic External Defibrillator) is an important device for primary help in case of cardiac arrest. It is vital that the AED is protected, secured and available 24/7 for immediate assistance in case of an emergency.

### Operator/ user

The person who is responsible for the cabinet and AED. His/ her duty is to check the status of the AED and Cabinet and assure operation complying to the directives as described in this manual.

### Online service

For more information or online help, please visit our website [www.sixcase.com](http://www.sixcase.com).

**User Manual:** The manual contains all necessary information about operating the SmartCase. Updates are published on our website.

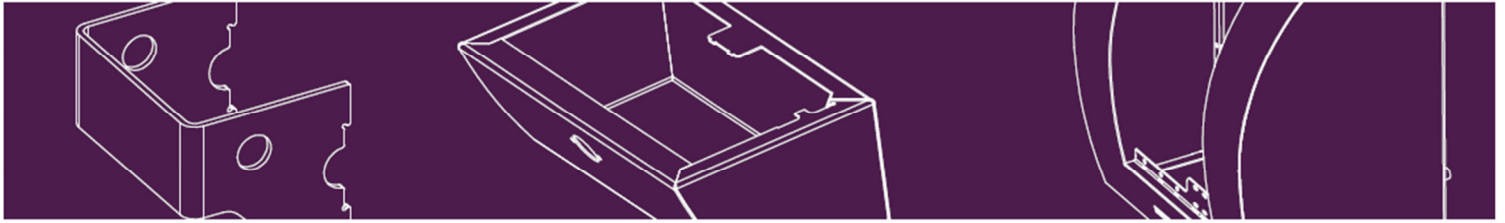
**Disclaimer:** GT MEDICARE is in no way responsible for the availability or serviceability of the AED stored in the SmartCase cabinet. GT MEDICARE is neither responsible for the installation, use or operation of the cabinet/AED nor for the loss of data, income, direct or indirect nor for caused damage to third parties. Exceptional external conditions, such as low temperatures, defect parts or other external elements may result in unserviceable AED's. GT MEDICARE preserves the right to adapt and to improve this document and all related products in this user manual, without preceding notification.

WARNING

### Contact


GT Medicare BV  
Industrieweg 46  
2382 NW Zoeterwoude  
The Netherlands  
T +031 (0) 71 2100 229  
E [info@sixcase.com](mailto:info@sixcase.com)

## Appendix A: Technical specification


### Technical Specification

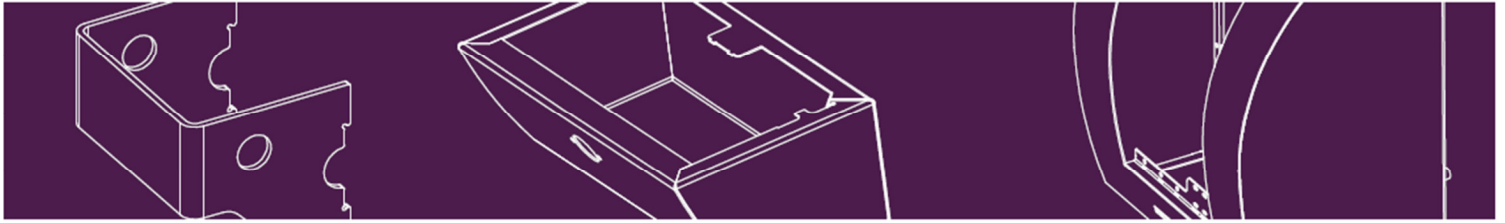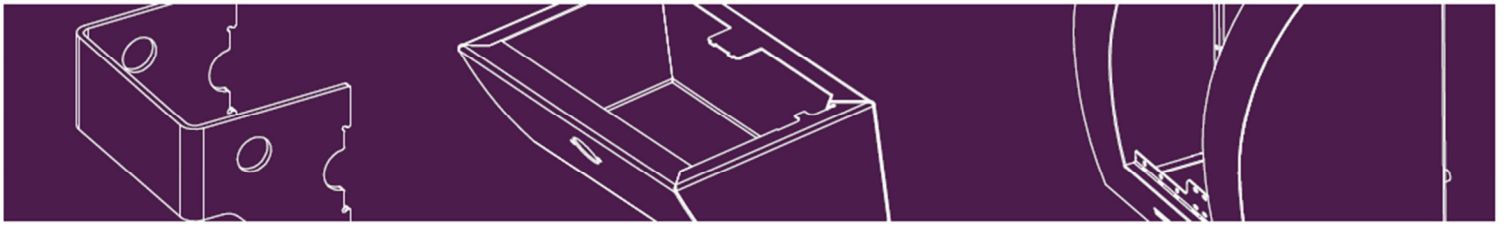Type	SC1230 / SC1240
Weight	9 kg
Dimensions	40(h)x40(w)x20(d)cm
Shockproof	IEC61439-5 clause 10.2.101.2/3
Waterproof	IP54
Cooling	Active ventilation
Heating	Electrical until -20 °C outside temperature
Corrosion resistance	Stainless steel with coating
Window	Impact proof polycarbonate
Installation kit	Included (wood & stone)
Power	24V Adapter (included)
Opening	SC1230: Door knob (90 degree turn) SC1240: Mechanical lock, 3 digits (independent of power)
Lighting	LED red en blue
Acoustic alarm	+/- 95dB intermittent

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